

Ice Cells

Professional's Choice®

Description:

These non-toxic Ice Cells offer a great way to effectively ice the legs after riding, or for therapy. And they are easy to use. Designed specifically for use with Professional's Choice Sports Medicine Boots, they fit securely inside the boot to evenly distribute cold therapy to your horse's leg. Plus, they remain flexible while frozen and retain their shape as they melt. Our Ice Cells are reusable and can be stored in the freezer. They also work great for icing down injuries sustained by the rider.

Features and Benefits:

Contains individual Liquid Chambers—Remains flexible when frozen to conform to horse's leg.

Reusable—Lasts for many uses, unlike ice.

Shaped to fit in SMB II's—Convenient because you use them with the boots you already have.

30 day Money Back Guarantee/Six Month Warranty

Ice Cell Application Instructions:

1. Place Ice Cells in the freezer until they are frozen. The Ice Cells may be stored in the freezer until they're needed.
2. Place the Ice Cells next to the skin on the leg area to be treated.
3. Position the Sports Medicine Boot over the Ice Cell to hold it in place.
4. Adjust the compression of the protective boots to suit your specific needs.

Washing Instructions:

Hand wash in cool water with a mild soap. Rinse thoroughly to remove any soap residue and allow to drip dry. Machine washing is not covered by the manufacturer's warranty.

Note—Do not use solvents or other chemicals on Professional's Choice products.